

**Contact**

eric@products4doctors.com

Call or Text 727-645-7774

ALLERGY 1·2·3™

PREVENT  
DISEASE

Prevent allergy-related  
diseases like asthma,  
chronic sinusitis  
& more

INNOVATIVE  
NEW TECH

Designed for the PCP

TURN-KEY  
EASY-TO-USE  
SOLUTION

Learn procedures  
to treat & prevent  
disease in a  
GROWING  
POPULATION  
OF ALLERGY  
SUFFERERS

MAKE  
MONEY  
LIKE A  
SPECIALIST




## QUALITY 15 MINUTE TEST KITS

- Steel lancet tips, not plastic
- High quality extracts
- Reliable, fast results

## CLEAR INSTRUCTIONS

- Test for 60 most common local airborne allergens

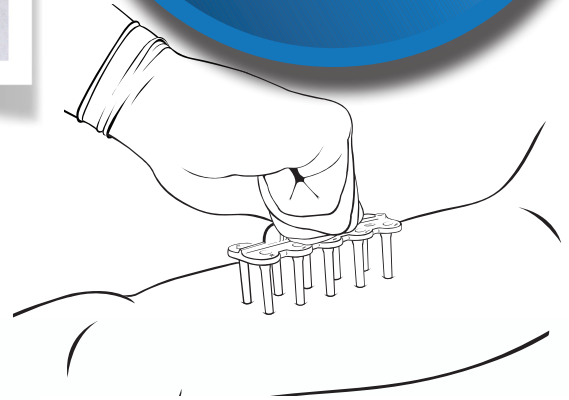

ALLERGY  
1-2-3™

### ALLERGY TEST KIT ASSEMBLY \*tests up to 20 patients

- 1 Wear sterile gloves. Be careful not to spill tray.
- 2 Remove black stoppers from each reservoir and discard.
- 3 Place white spacers upright in rectangular slots.

- 4 Open one patient test kit and remove Comforten steel lancet from packaging. Set aside patient box for last (20th) test.
- 5 Lining up the lancet arrow with the alignment dot in the tray, place lancet in reservoir. Repeat with all 6 lancets.
- 6 Replace acrylic lid over entire test kit. Refrigerate.




# SMART TECHNOLOGY

- Easy to use Ipad App
- Smart App suggests treatment set based on patient history, prevalence, local bloom times & more

## ONE CLICK ORDERING

Easy to order custom patient supplies

### PRIMARY AND SECONDARY IMMUNE RESPONSE

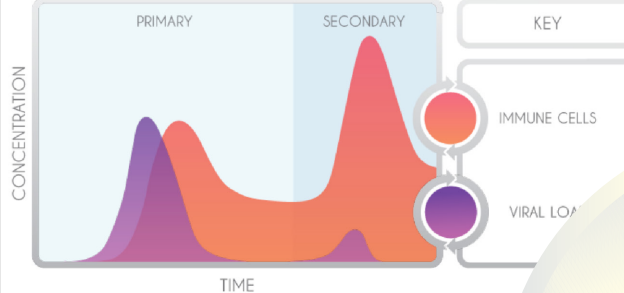

## ONLINE TRAINING

Training available for the entire team

- Clinical
- Front Desk
- Management
- Billing

Contact  
eric@products4doctors.com  
Call or Text 727-645-7774


## EASY CUSTOM MIXING KIT

- Mix patient's custom immunotherapy in 2-3 min
- Compliant with all federal and state regulations



### INJECTION FLOW CHART

Use this question flow chart as a guide BEFORE administering any immunotherapy doses.

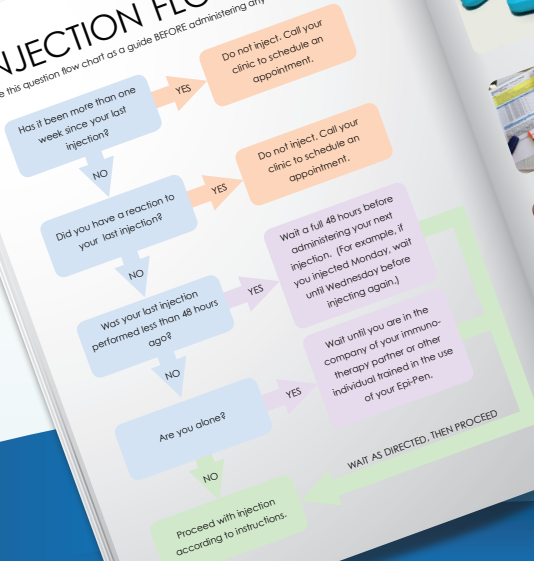

### INJECTION INSTRUCTIONS

- 1 Take an antihistamine as directed by your doctor regularly and before any injection. Wait 15 min to take effect.
- 2 Gather all supplies: Dosage Schedule Epi-Pen Alcohol wipes Vial of serum Unused needle Safe Clip Container
- 3 Clean top of serum vial thoroughly with an alcohol swab.
- 4 Clean injection site with alcohol. Alternate between your body and your dosage vial. If you don't know the right way to do this, ask your doctor.
- 5 Remove the cap from the end of the needle.
- 6 Remove cap from the plunger end of the needle.

## SIMPLE PATIENT INSTRUCTIONS

- Simple instructions for at-home allergy shots
- Booklet contains all safety training, injection schedules, waivers

[www.products4doctors.com](http://www.products4doctors.com)

727-999-3119